# TSG 130

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## Table of contents

Preface .....	7
Declaration of conformity .....	7
Service life .....	7
Disposal .....	7
Identification plate .....	7
Reuse.....	7
Warranty .....	7
Parts of the product .....	8
Indication.....	8
Scope of delivery .....	8
Safety instructions.....	9
Height settings .....	9
Assembly.....	9
Cleaning   Maintenance.....	9
Technical data.....	9

## Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

## Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

## Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.

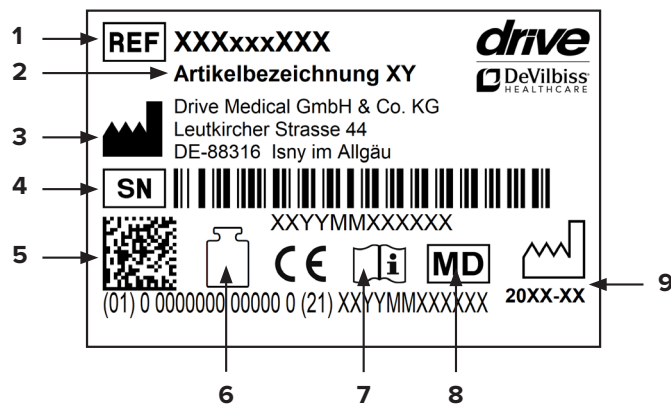
This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

## Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

## Identification plate

1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

## Reuse

This product can be reused. Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at [www.drivedevilbiss.de](http://www.drivedevilbiss.de)

## Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

### EXCLUSION OF WARRANTY

Parts subject to wear include (among others): rubber feet, armrests

## Parts of the product

TSG 130

- |                                                                                                                                                                                                              |                                                                                               |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------|
| <ul style="list-style-type: none"> <li>1. Backrest</li> <li>2. Armrest</li> <li>3. Toilet lid, removable</li> <li>4. Toilet seat, removable</li> <li>5. Toilet bucket with cover and splash guard</li> </ul> | <ul style="list-style-type: none"> <li>6. Height-adjustable feet</li> <li>7. Frame</li> </ul> |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------|

### Indication

#### INDICATION:

Use of the toilet hindered due to unadapted seat height through limited ability to sit down and stand up along with limited control of the trunk with functional disorder of the lower extremity with significantly restricted or fully absent movement of the joints and/or limited muscular strength of the lower extremity and/or damage to the lumbar spine, hip and/or knee and/or damage to the nervous system/central nervous system and/or damage to muscle function with limited or absent stability of the trunk.

### Scope of delivery

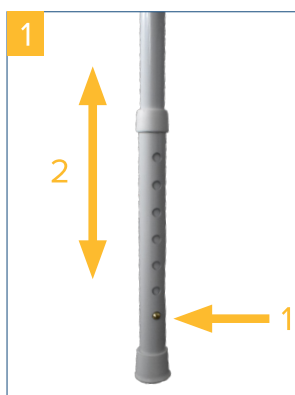
<b>1 x</b>	commode, fully assembled
<b>1 x</b>	splash guard
<b>1 x</b>	bucket
<b>1 x</b>	instructions for use

## Safety instructions

- Do not use if there are any defects!
- Caution – risk of slipping/falling!
- The lid must not be used as a backrest in the opened position!
- The lid must not bear a load when closed (by sitting on it, for example)!
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the type label.

## Height settings

Before using the product, adjust it to the desired height. To adjust the height of the legs: Press button **(1)** and at the same time, pull the leg out or push it in until the button clicks into place in the desired position **(2)**. Ensure that the pushbutton has correctly clicked into place **(photo 1)**.

Ensure that all four legs are adjusted to the same height.

## Assembly

The product is preassembled; only the seat and the lid have to be clipped into place.

### Bucket

To insert the bucket, fold the lid and seat upwards. Then place the bucket from above onto the holder. To remove the bucket, pull it up and out using the carrying handle. **(photo 2)**.

### Splash guard

Instead of the toilet bucket, the splash guard can also be inserted if used over a conventional toilet **(photo 3)**.

## Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60°C for long periods of time (for example, use in a sauna, etc.).
- In the event of defects, contact your specialist dealer from whom you purchased this product.

## Technical data

Product	TSG 130
Article number	500800100
HMV (medical aids register) no.	33.40.02.1013
Seat depth	45 cm
Seat width	44,5 cm
Overall height	60 - 75 cm
Overall length	52 cm
Overall width	67 cm
Weight	5,7 kg
Max. load	130 kg
Material	Plastic (PP / PA)
Material frame	Aluminium

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Drive Medical GmbH & Co. KG